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EQUATOR  
COMPONENT  
**English**

COMPONENTE  
EQUATOR  
**Español**

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### INSTRUCTIONS FOR USE

---

TECHNICAL NAME: Dental Implant Components

TRADE NAME: Equator Component

MODEL: For a detailed description of the product, see tables 1 and 2.

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E-mail: [customerservice@plenum.bio](mailto:customerservice@plenum.bio)

Website: [www.plenum.bio](http://www.plenum.bio)

ATTENTION: Carefully read these Instructions for Use prior to using the product. Follow the recommendations, warnings and precautions mentioned in these Instructions for Use. Failure to observe these points may lead to complications.

Product for Dental Use.

Product Supplied without Sterilization.

Expiration date: See device labeling.

Single-use only. Prohibited reprocessing and reuse.

ANVISA REGISTRATION: 81684349003

Device code, batch code, and date of manufacture: See device labeling.

REV. 01

## SUMMARY

1. DETAILED DEVICE DESCRIPTION	3
2. DEVICE COMPOSITION	4
3. DEVICE PRESENTATION FORM	4
4. DEVICE RECOMMENDATIONS, PURPOSE, OR INTENDED USE	4
5. PRINCIPLES OF INTENDED DEVICE FUNCTION AND PERFORMANCE	5
6. INSTRUCTIONS FOR USE	5
7. PRECAUTIONS	5
8. WARNINGS	6
9. CONTRAINDICATIONS	6
10. POSSIBLE ADVERSE EFFECTS	6
11. SYMBOL DESCRIPTIONS	6
12. AUXILIARY INSTRUMENTS	7
13. CLEANING AND DISINFECTION	7
14. EXPIRATION DATE	7
15. IDENTIFICATION AND TRACEABILITY	7
16. STORAGE, TRANSPORT, AND HANDLING CONDITIONS	7
17. DISPOSAL OF DEVICE	8
18. CUSTOMER SERVICES	8
19. LIMITED WARRANTY	8

# 1. DETAILED DEVICE DESCRIPTION

## ACCEPTABLE METAL COMPOUNDS

Equator Components are designed to be installed next to the Equator Abutment (the Equator Abutment is not part of this family and must be purchased separately) and serve as fixation and retention elements in complete removable implant/mucous-supported prostheses.

The list of “Acceptable Metal Compounds” was based on ISO 21534. There is, therefore total compatibility between the raw materials used in the manufacturing of Equator Component models and Plenum Dental Implants.

Tables 1 and 2 below present the illustrative images, codes, models, and main dimensions of Equator Components.

**TABLE 1.** Comparative table for Equator components, models housings, and caps.

Illustrative Image	Code	Model	Model Content	Main Dimensions (mm)	Raw material
	018-20-C15-00-03	FLAT HOUSING	FLAT HOUSING+ PROCESSING CAP + PROTECTIVE DISK	Ø4.5 x 2.0	Ti-6Al-4V ELI
	018-20-C15-00-04	ANGULAR HOUSING	ANGULAR HOUSING + PROCESSING CAP + PROTECTIVE DISK	Ø4.5 x 2.2	Ti-6Al-4V ELI
	018-20-C15-00-05	STRONG CAP	STRONG CAP	Ø3.8 x 1.7	Polyoxymethylene
	018-20-C15-00-06	STANDARD CAP	STANDARD CAP	Ø3.8 x 1.7	Polyamide
	018-20-C15-00-07	SOFT CAP	SOFT CAP	Ø3.8 x 1.7	Polyamide
	018-20-C15-00-08	EXTRA SOFT CAP	EXTRA SOFT CAP	Ø3.8 x 1.7	Polyether

**TABLE 2.** Comparative table of replacement items.

Illustrative Image	Code	Model	Model Content	Main Dimensions (mm)	Raw material
	018-20-C15-00-09	COIFA PROTÉTICA	COIFA PROTÉTICA	Ø3,8 x 1,7	Polioximetileno
	018-20-C15-00-10	DISCO DE PROTEÇÃO	DISCO DE PROTEÇÃO	Ø 14 x 1	Acetato-vinilo de etileno

The models of caps and protective disks presented are compatible with the flat and angular housing models presented in table 1.

## 2. DEVICE COMPOSITION

Flat and Angular Housing models are manufactured using Titanium Alloy Ti-6Al-4V ELI, according to the specifications contained in ASTM F136.

The models and replacement items Processing Caps and Protective Disks are manufactured in polymer, according to ISO 1183, ISO 527 and ISO 178.

NOTE: The specific polymers used include: polyoxymethylene to STRONG CAP (purple), polyamide to STANDARD CAP (transparent) and SOFT CAP (Pink), polyether to EXTRA SOFT CAP (yellow), and ethylene acetate-vinyl to PROTECTIVE DISK.


## 3. DEVICE PRESENTATION FORM

Equator Components according to the “composition of models” described in the tables above, are supplied packaged according to the contents of their respective set, in non-sterile condition and presented as follows:


- Primary packaging is a polyethylene terephthalate (PET) blister and thermo-sealed with surgical grade paper; containing:
  - 1 device set (housing + processing cap + protective disk, see table 1) or;
  - 4 cap units (purple, transparent, pink or yellow) or;
  - 4 units of replacement parts for Processing Caps or Protective disks (table 2);
- External packaging is a rigid cardboard box containing the primary packaging and three (03) identification and traceability labels;
- The hospital/health service establishment is responsible for guaranteeing the complete traceability of the device through means of annotation in the patient’s medical records, the code and lot of the device used, or identification through one of the three (03) traceability labels that accompany the packaging;
- A printed version of the Instructions for Use (including shipping), can be requested at no additional cost at the telephone number +55 (11) 3109-9000, or through the electronic address ([customerservice@plenum.bio](mailto:customerservice@plenum.bio)) or through the website ([www.plenum.bio/ifu](http://www.plenum.bio/ifu));
- Illustrative images of the primary and external packaging of available commercialized Equator Component models can be found below.

NOTE: There will be 2 types of primary packaging provided (1: housing + processing cap + protective disk or 2: caps/protective disks for replacement).

ILLUSTRATIVE IMAGE OF PRIMARY PACKAGING OPTIONS FOR EQUATOR COMPONENTS

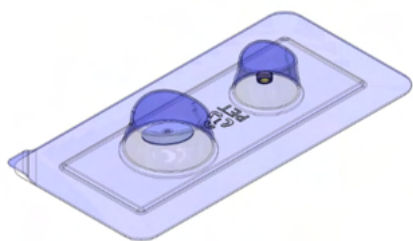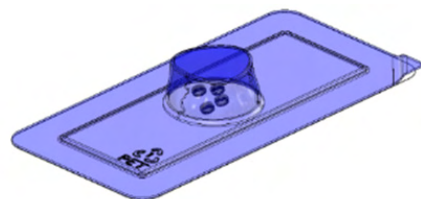1 - Housing + processing cap + protective disk


2 - Caps (4 units) and Processing Caps or Protective disks for replacement.

ILLUSTRATIVE IMAGE OF THE EQUATOR COMPONENT’S EXTERNAL PACKAGING

## 4. DEVICE RECOMMENDATIONS, PURPOSE, OR INTENDED USE

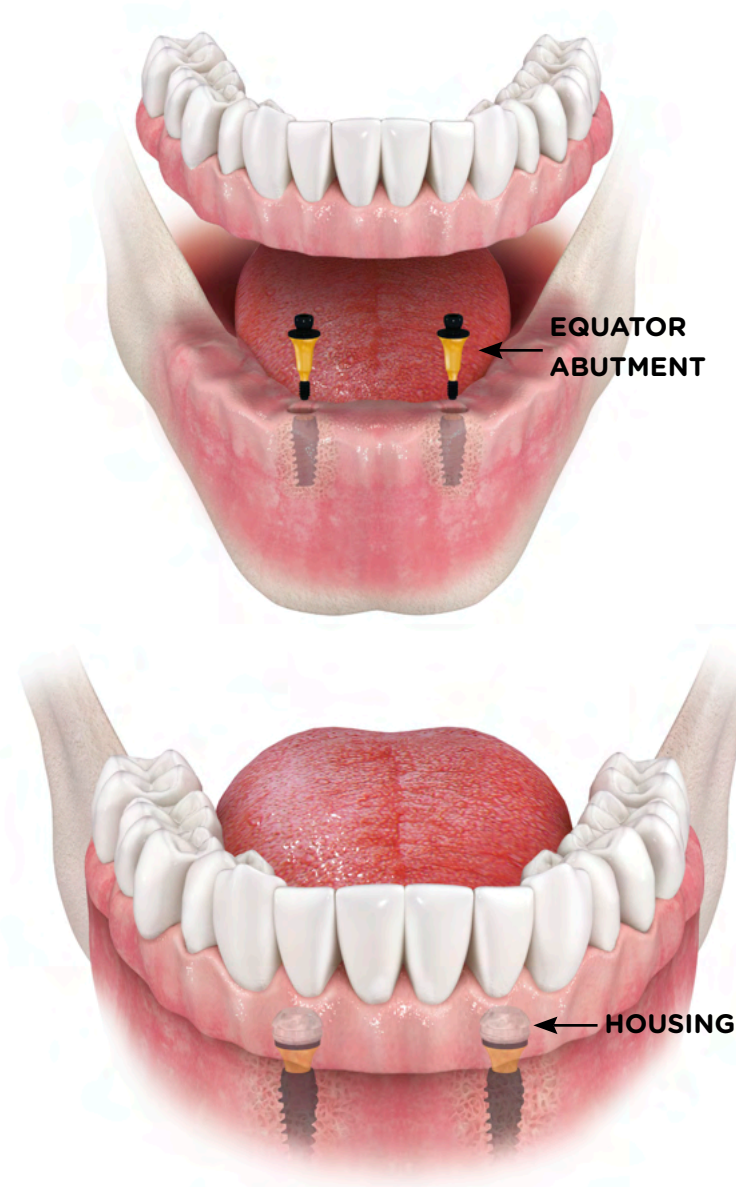Equator Components are designed to be installed next to the Equator Abutment and serve as fixation and retention elements in complete removable implant/mucous-supported prostheses.

ATTENTION: Figures are merely illustrative. They do not represent the actual dimensions and characteristics of the product.

## 5. PRINCIPLES OF INTENDED DEVICE FUNCTION AND PERFORMANCE

Housings are fixed to the complete prosthesis (denture) and serve to guarantee adaptation between the caps and the Equator Abutment.


Polymeric caps are attached to the housings and serve to retain the prosthesis and provide cushioning against masticatory impacts.

Processing caps are used to protect the interior of the housing during capture and molding procedures.

The Protective Disk protects the Equator Abutment during procedures and capture and molding.

## 6. INSTRUCTIONS FOR USE

- Install the Equator Abutment on the Plenum Implant (Figure 1);
- Install the Protective Disk over the Equator Abutment (Figure 2);
- Install the Processing Caps (black) on the Flat or Angular Housing using a Driver.

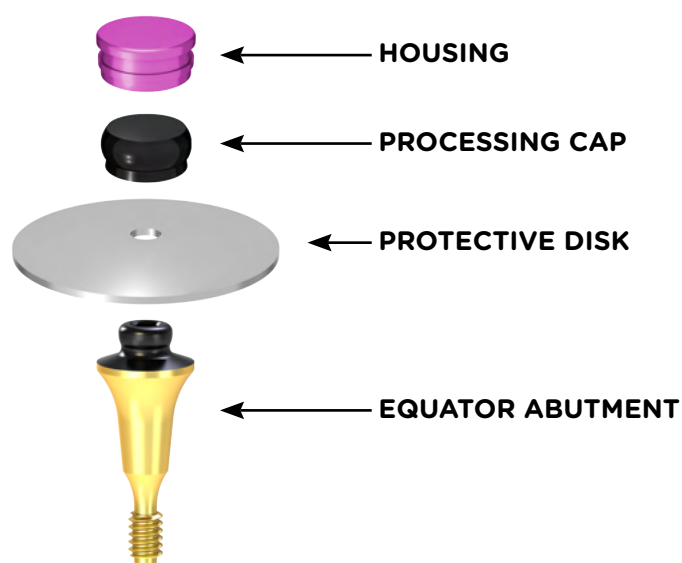
**FIGURE 1:** Sequence of use of the Equator Abutment, Protective Disk, Processing Cap and Flat Housing during the capture or molding procedure.

- Pour the acrylic resin into the previously prepared prosthesis using the corresponding niches for each Equator Component. Insert the prosthesis into the Housing and wait for polymerization to occur;;
- After complete polymerization of the resin, remove the prosthesis. The Housing and Processing Cap will be attached to the prosthesis. Using the EQUATOR's INSERTER/EXTRACTOR Driver, remove the Processing Cap (black) and install the selected Cap according to the desired retention level (see models in table O2):

Cap/Degree of Retention	Color
Strong Cap	Purple
Standard Cap	Transparent
Soft Cap	Pink
Extra Soft Cap	Yellow

- Remove the Protective Disk and any excess resin on the Equator Abutment or soft tissue;
- Install the prosthesis and verify the need for additional occlusal adjustments.

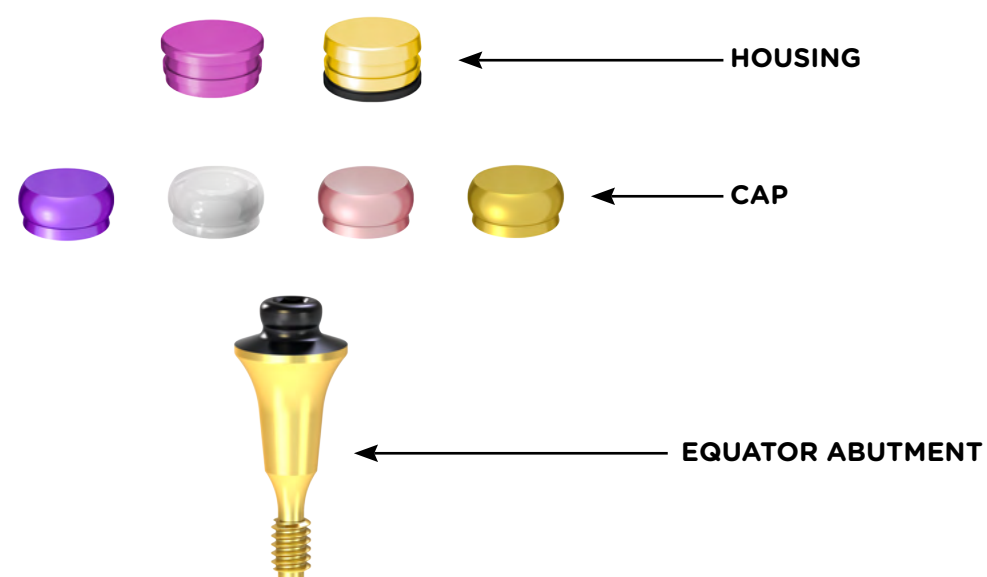**FIGURE 2:** Sequence of use of the Equator Abutment, Cap and Housings.

## 7. PRECAUTIONS

- Dentists must not use this device until they have carefully read the instructions for use and the information provided on the packaging. A failure to observe these points may result in complications during procedures;
- The Equator Components that are part of this family must be cleaned and disinfected before use (see item Cleaning and disinfection);
- This family of devices has a maximum useful life (use of the device after prosthesis installation) of 12 months. Therefore, we recommend changing the housing, cap, and protective disc assembly within a maximum period of 12 months;
- Immediate and subsequent postoperative care and proper oral hygiene techniques are extremely important. The patient must be given instructions regarding the existing risks for the maintenance of implants and components;
- The dentist must clearly guide the patient, who must understand the importance of follow-up during the postoperative period, in which the stability, integrity, and positioning of the dental implant, component, and implant/mucous-supported prosthesis must be verified;
- The patient must be warned that a failure to comply with instructions for the post-operative period (immediate and subsequent stages) may lead to failure and early loss of the component and implant, requiring additional surgery to remove the implant or replace the component;
- Prosthetic components and implants manufactured by Plenum must be exclusively used in procedures to guarantee the safety and effectiveness of treatment; Components and/or implants manufactured by other companies may result in dimensional incompatibility, improper fitting, micro movements and increase the risk of fracturing in these products, compromising the results of the surgical and prosthetic procedure.
- Incorrect handling of instruments with small dimensions inside the patient's mouth may present a risk of aspiration and/or swallowing of these elements;
- If deformations, scratches, cracks, or visible dents are found in the component, it must be discarded;

- Patients must be informed of all limitations, risks, complications, warnings, and potential adverse effects arising from the procedure for installing the dental implant and prosthetic component set;
- The patient must be informed of the risks involved if the dental implant and prosthetic component assembly is overloaded beyond its functional capacity, which may cause bone loss, failure of the implant, its components, and prosthetic restoration;
- It is important to ensure complete traceability of the component used through the annotation in the patient’s chart of the code and lot of the device used or by fixing the identification and traceability labels that accompany the product in the patient’s records to protect the patient and their dental surgeon;
- Care must be taken with the Cap’s inner surface and the Equator Abutment’s spherical connection. Both surfaces must be free of particles and solid materials that can prevent perfect fitting and fastening between the Housing and Equator Abutment, thereby preventing the proper fixation of these elements;
- Regular monitoring of the patient is recommended for the purpose of maintaining the prosthesis and caps;
- It is recommended that caps be replaced annually or whenever there is a loss or decrease in the retention of the prosthesis;
- Elderly patients, patients with mental health problems, individuals with chemical dependency, or any other difficulty in adhering to postoperative recommendations may represent a higher risk of failure of this product since these patients may ignore instructions from the dental surgeon for this procedure.

## 8. WARNINGS

- Single-use only, do not reuse.
- The Equator Components that are part of this family are supplied in non-sterile conditions, and should not be sterilized by autoclaving;
- Do not use if the package is opened or damaged.

## 9. CONTRAINDICATIONS

- Equator Components are always used in conjunction with Plenum’s Equator Abutments and Dental Implants (supplied separately). Dentists must therefore assess the patient’s condition prudently prior to surgery and use of the components, and check for any of the contraindications reported below. Contraindications may include, but are not limited to:
  - Patients that present signs of allergy or hypersensitivity to the composition of the raw materials used in manufacturing of the components (see “Composition”);
  - An implant that is poorly positioned and/or presents insufficient osseointegration to properly support or anchor a dental prosthesis (crown) and its components, or is subject to strong restrictions on the use of this product;
  - Angulation between implants greater than 25°;
  - Patient with poor bone quality due to osteoporosis or osteopenia, which can cause early failure of the dental implant and component due to lack of osseointegration;

- Insufficient interocclusal or mesiodistal space for prosthetic rehabilitation;
- Inflammation and/or periodontal infection;
- Recent history of systemic or localized infection;
- Parafunctional habits;
- Non-treatable occlusion and joint problems;
- Use in patients with immunological changes that result in poor healing or a lack of integrity in existing tissue at the implant site;
- Immunosuppressive pathologies;
- Patient undergoing radiotherapy or using medications that compromise the vascularization of bone and soft tissues;
- Patients that are alcoholics or drug users and cannot correctly follow postoperative instructions;
- Patients with a mental illness or that are unable to follow recommendations and care during the postoperative phase;
- Patients that are unable to maintain proper oral hygiene;
- All contraindications included in known dental and scientific literature for the use of dental implants and their components.

## 10. POSSIBLE ADVERSE EFFECTS

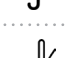
Adverse effects are not expected as long as the product is used in accordance with its instructions for use.

## 11. SYMBOL DESCRIPTIONS

Table 3 describes the symbols that may be on the device labeling. Please check the physical labeling for the symbols applied to the device. The symbols express information related to the identification and use of the device.

**TABLE 3.** Symbols used for labeling to express information related to the identification and use of the device.

	Manufacturer
	Country and date of manufacture
	Use-by date
	Batch code
	Catalog number
	Sterilized using ethylene oxide
	Sterilized using Irradiation
	Sterilized using vaporized hydrogen peroxide
	Non-sterile

	Do not re-sterilize
	Do not use if package is damaged
	Single sterile barrier system
	Double sterile barrier system
	Single sterile barrier system with protective packaging inside
	Single sterile barrier system with protective packaging outside
	Keep away from sunlight
	Keep dry
	Lower limit of temperature
	Upper limit of temperature
	Temperature limit
	Do not re-use
	Consult electronic instructions for use (plenum.bio/ifu)
	Unique device identifier
	Device manufacturing material
	Caution: Federal law restricts this device to sale by or on the order of a physician or dentist. (Notification required by FDA for United States market)
	Reprocessing prohibited (Brazilian Requirement)

**NOTE:** The symbols related to the labeling meet the requirements established in the ISO 15223-1:2021 standard.

## 12. AUXILIARY INSTRUMENTS

EQUATOR INSERTER/EXTRACTOR driver, Ref. 018-15-F20-00-30. (not included in these instructions for use and must be purchased separately).

## 13. CLEANING AND DISINFECTION

- Since they are made of polymers that are sensitive to high temperatures, it is not recommended that caps and protective discs be sterilized in an autoclave since exposure to the temperatures inherent to this sterilization process would result in deformation of the parts and possible dimensional incompatibility.

- It is recommended that the housings, caps, and disks be placed in an ultrasonic bath with water and enzymatic detergent at room temperature for 5 minutes. Once the bath is finished, the pieces must be thoroughly rinsed in running water, dried with disposable paper towels, and then fully immersed in 70% alcohol for a period of 30 minutes.

- It is not recommended that this product be used without completing the step described above.

## 14. EXPIRATION DATE

- Indicated on device labeling.
- The device has an unexpired expiration date.

## 15. IDENTIFICATION AND TRACEABILITY

To ensure the traceability of Equator Components and comply with health surveillance requirements, the dentist or someone on their team must keep information about the component used in the patient's medical record. Additionally, this information must also be passed on to the product distributor and patient, in order to complete the traceability cycle. The information necessary for traceability is related to the component used and surgical procedure, and patient, as follows:

- Name of dentist;
- Name of hospital/clinic;
- Name of manufacturer and distributor;
- Date of surgery;
- Code of component used and component batch number;
- Quantity used;
- Product registration with ANVISA.

The responsible dentist and their team must make use of the information contained in the 03 traceability labels supplied with the component, fixing this information in the patient's clinical record to maintain traceability. In addition, this information must be provided to the patient so that they have information about the product used in their procedure.

Labels contain the following information necessary for the traceability of the prosthetic component used.

- Manufacturer identification;
- Product name or commercial model;
- Product code and batch number;
- ANVISA registration number.

Traceability information is necessary for notification by the health service and/or by the patient to the National Health Regulatory Agency - ANVISA and to the manufacturer, when complaints, technical complaints and/or adverse events occur, for the carrying out of the appropriate investigations.

## 16. STORAGE, TRANSPORT, AND HANDLING CONDITIONS

- Packaging must be intact at the moment at which it is received. Do not use the device if the packaging is damaged or tampered with. If the packaging is damaged, the device must be discarded;

- The device must be stored and transported in a clean environment, away from sources of heat or humidity, sunlight and the action of the weather in order to prevent any damage or changes in the packaging or its physical form that renders its use impossible;
- The device must be stored on shelving or drawers/ cabinet in a manner that allows it to be easily viewed and handled by the operator. Do not store the device near lightbulbs to ensure that the information contained in the manufacturer's packaging is not affected;
- The handling of the device should only be done by qualified and trained personnel to perform this procedure.

## 17. DISPOSAL OF DEVICE

- Discard the device if any of the series of the packaging is damaged or opened;
- After use of the device or their being removed from patient, Equator Components that are not subject to analysis or subsequent studies must be disposed of by the health service establishment (hospitals and clinics);
- It is recommended that devices be completely disassembled to ensure that they cannot be reused. The health service institution/establishment is solely responsible for disassembling the device;
- After disassembly, the instrument must be identified as being unsuitable for use and disposed of in accordance with health service waste management standards (hospitals and clinics) and applicable environmental legislation;
- Components that are rendered useless during the procedures must also be discarded following the guidelines described above.

## 18. CUSTOMER SERVICES

- If the device presents any adverse event with a risk that may harm the patient, generates or has the potential for injury or threat to public health, is out of its specifications, or is causing any dissatisfaction, the health professional must generate a technical report containing the details of the non-compliance, with all the claimant data, device code, batch code, and patient data; directly notifying the Plenum Customer Service;
- If necessary, send the clean, decontaminated, sterilized, packaged, and properly identified device, with the description of non-conformities, to the following address: M3 Health Indústria e Comércio de Produtos Médicos, Odontológicos e Correlatos S.A. Avenida Ain Ata, N° 640 - Lote 15 - Quadra B - Jardim Ermida I, Multivias II - Polo Industrial e Logística - CEP 13212-213 - Jundiaí - SP - Brazil. Phone: +55 11 3109-9000; E-mail: [customerservice@plenum.bio](mailto:customerservice@plenum.bio)
- ANVISA also provides a channel to receive notifications of incidents, adverse events, and technical complaints related to the use of devices and services under health surveillance, which can be found on the National Health Surveillance Agency website - ANVISA, at [www.anvisa.gov.br](http://www.anvisa.gov.br), link NOTIVISA.
- FDA also provides a channel available to report incidents, adverse events, and technical complaints related to the use of devices and services subject to health regulations, which can be found in the FDA Manufacturer and User Facility Device Experience (MAUDE) database.

## 19. LIMITED WARRANTY

Devices manufactured by Plenum are provided with a factory warranty protecting against manufacturing defects. Device defects must be immediately reported to the manufacturer in adherence to legally established deadlines, with an attached technical report detailing the non-compliance identified in the device. This report must be prepared and signed by a qualified health professional and addressed to regulatory authority standards (see the Customer Services section) when it is observed that issues involve a risk to the patient's health.

The warranty of devices manufactured by Plenum is conditioned to the strict observation and follow-up of the indications for use, contraindications, precautions, warnings, and other information contained in the instructions for use. The lack of such compliance and monitoring will exempt the manufacturer and distributor from any form of liability, particularly in the following cases:

- Reuse of the device by the user;
- Improper use or use other than in a surgical environment;

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