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TRANSFER
English

TRANSFERENTE
Español

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Transfer

INSTRUCTIONS FOR USE

TECHNICAL NAME: Dental Implant Components

TRADE NAME: Transfer

MODEL: For a detailed description of the device, see Table 1.

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E-mail: customerservice@plenum.bio

Website: www.plenum.bio

ATTENTION: Carefully read these instructions for use before using the device. Follow the recommendations, warnings, and precautions mentioned in these Instructions for use. A failure to observe these items may result in complications.

Device for Medical and Dental use. Sterile Device. Sterilized using Gamma Irradiation. Expiration date: See device labeling. Single-use only. Prohibited reprocessing, resterilization and reuse.

ANVISA REGISTRATION N°: 81684340010.

Code, lot, and date of manufacture and sterilization: See device labeling.

SUMMARY

1. DETAILED DEVICE DESCRIPTION	3
2. DEVICE COMPOSITION	4
3. DEVICE PRESENTATION FORM	4
4. DEVICE RECOMMENDATIONS, PURPOSE, OR INTENDED USE	4
5. PRINCIPLES OF INTENDED DEVICE FUNCTION AND PERFORMANCE	4
6. INSTRUCTIONS FOR USE	4
7. PRECAUTIONS	6
8. WARNINGS	6
9. CONTRAINDICATIONS	6
10. POSSIBLE ADVERSE EFFECTS	6
11. SYMBOLS DESCRIPTION	6
12. STERILIZATION	7
13. EXPIRATION DATE	7
14. IDENTIFICATION AND TRACEABILITY	7
15. STORAGE, TRANSPORT, CONSERVATION AND HANDLING CONDITIONS	8
16. DISPOSAL OF DEVICE	8
17. CUSTOMER SERVICES	8
18. LIMITED WARRANTY	8

1. DETAILED DEVICE DESCRIPTION

The Transfer is a metallic product that has a cylindrical external body, milled, of transitory use, whose objective is to transfer the correct position of the

implant and/or the prosthetic component, previously installed in the patient, to a plaster model so that the axis of insertion of the prosthesis, which will be made, is consistent with that of the selected prosthetic abutment.

Illustrative images, codes, models, prosthetic platforms, main dimensions, and a summary of the function of Transfers are presented in Table 1.

TABLE 1. Illustrative images, codes, models, prosthetic platforms and main dimensions of transfers.

Illustrative image	Code	Name/Model commercial	Content	Diameter (mm)	Length (mm)	Prosthetic Interface
	018-21-C30-00-00	IMPLANT SL-OPEN TRAY	Transfer + Transfer Open Tray Screw	3.0		SL
	018-22-C30-00-00	IMPLANT RE-OPEN TRAY		3.5		RE
	018-23-C30-00-00	IMPLANT SH-OPEN TRAY		3.5		SH
	018-20-C30-50-33	ABUTMENT Ø3.5X4mm-OPEN TRAY		3.5	4	
	018-20-C30-51-33	ABUTMENT Ø3.5X6mm-OPEN TRAY		3.5	6	
	018-20-C30-50-45	ABUTMENT Ø4.5X4mm-OPEN TRAY		4.5	4	
	018-20-C30-51-45	ABUTMENT Ø4.5X6mm-OPEN TRAY		4.5	6	
	018-20-C30-20-00	MULTI ABUTMENT-OPEN TRAY	Transfer + Coping Screw	4.5		
	018-21-C31-00-00	IMPLANT SL-CLOSED TRAY	Transfer + Abutment Screw + clip*	3.0		SL
	018-22-C31-00-00	IMPLANT RE-CLOSED TRAY		3.5		RE
	018-23-C31-00-00	IMPLANT SH-CLOSED TRAY		3.5		SH
	018-20-C31-50-33	ABUTMENT Ø3.5X4mm-CLOSED TRAY		3.5	4	
	018-20-C31-51-33	ABUTMENT Ø3.5X6mm-CLOSED TRAY		3.5	6	
	018-20-C31-50-45	ABUTMENT Ø4.5X4mm-CLOSED TRAY		4.5	4	
	018-20-C31-51-45	ABUTMENT Ø4.5X6mm-CLOSED TRAY		4.5	6	
	018-20-C31-15-00	EQUATOR		4.5		
	018-20-C31-20-00	MULTI ABUTMENT-CLOSED TRAY	Transfer + Coping Screw	4.5		
	018-21-C32-00-00	IMPLANT SL-DUO TRAY	Transfer + Abutment Screw+ Clip*+ Transfer Open Tray Screw	3.0		SL
	018-22-C32-00-00	IMPLANT RE-DUO TRAY		3.5		RE
	018-23-C32-00-00	IMPLANT SH-DUO TRAY		3.5		SH
	018-24-C32-00-00	IMPLANT UF-DUO TRAY		3.5		UF
	018-25-C32-00-00	IMPLANT MF-DUO TRAY		3.5		MF
	018-20-C32-50-33	ABUTMENT Ø3.5X4mm-DUO TRAY		3.5	4	
	018-20-C32-51-33	ABUTMENT Ø3.5X6mm-DUO TRAY		3.5	6	
	018-20-C32-50-45	ABUTMENT Ø4.5X4mm-DUO TRAY		4.5	4	
	018-20-C32-51-45	ABUTMENT Ø4.5X6mm-DUO TRAY		4.5	6	
	018-20-C32-20-00	MULTI ABUTMENT-DUO TRAY		Transfer + Coping Screw + Clip* +Transfer Open Tray Screw	4.5	
	018-20-C32-20-36	MULTI ABUTMENT-DUO TRAY Ø3.6		3.6		
	018-20-C32-35-04	UNIV.ABT.Ø3.5X4MM-DUO TRAY	Transfer + Clip	3.5	4	
	018-20-C32-35-06	UNIV.ABT.Ø3.5X6MM-DUO TRAY		3.5	6	
	018-20-C32-45-04	UNIV.ABT.Ø4.5X4MM-DUO TRAY		4.5	4	
	018-20-C32-45-06	UNIV.ABT.Ø4.5X6MM-DUO TRAY		4.5	6	
REPLACEMENT ITEMS						
	018-21-C32-00-01	TRANSFER CLIP SL	CLIP			SL
	018-22-C32-00-01	TRANSFER CLIP RE				RE
	018-23-C32-00-01	TRANSFER CLIP SH/SX				SH/SX
	018-24-C32-00-01	TRANSFER CLIP UF				UF
	018-25-C32-00-01	TRANSFER CLIP MF				MF
	018-20-C32-00-01	TRANSFER CLIP ABT.				

IMPORTANT:

- When available, the Transfers' colors represent the Prosthetic interface: PINK interface with SLIM prosthetic components; YELLOW interface with REGULAR prosthetic components; GREEN interface with SHORT prosthetic components. When not anodized (gray color), the Transfers' colors represent adaptations for the Prosthetic Components, such as: Existing copings represent adaptation to Components.
- The acronym MF corresponds to the closed tray molding technique.
- The acronym OT corresponds to the open tray molding technique.
- Transfers are compatible with all Plenum Implants, provided that the prosthetic interface corresponds to the implant's internal connection (SL, RE, SH/ SX, UF e MF).

2. DEVICE COMPOSITION

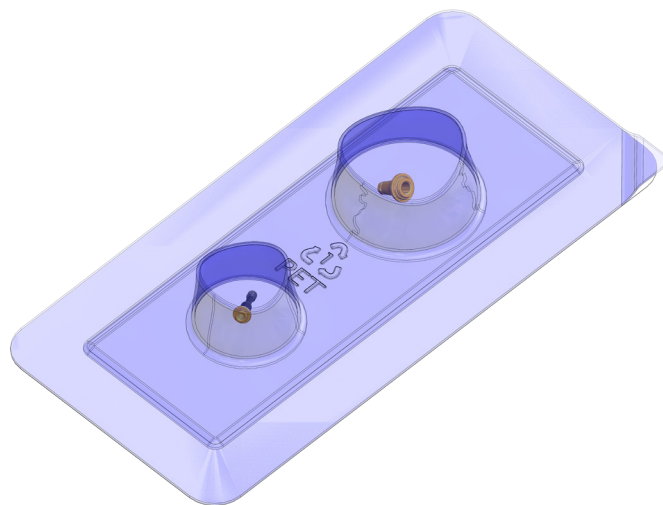
The devices that are part of the Transfer family are manufactured using Grade 4 Titanium, according to the specifications contained in ASTM F67 – Standard Specification for Unalloyed Titanium, for Surgical Implant Applications and ISO 5832-2 – Implants for surgery – Metallic materials Part 2: Pure titanium.

3. DEVICE PRESENTATION FORM

Transfers in the models described in the table above are supplied in a single package in sterile condition and presented in the following manner:

- Primary packaging is a polyethylene terephthalate (PET) blister and thermo-sealed with surgical grade paper; containing 01 unit of the device, sterilized using Gamma Irradiation;
- External packaging is a cardboard box containing the primary packaging and three (03) identification and traceability labels;
- It is the responsibility of the hospital/health care facility to ensure complete traceability of the device, through the annotation in the patient's chart of the code and lot of the device used, or identification through one of the three (03) traceability labels that come with the package;
- A printed version of the Instructions for Use (including shipping), can be requested at no additional cost at the telephone number + 55 11 3109-9000, or through the electronic address (customerservice@plenum.bio) or by the website (www.plenum.bio/ifu);
- See below the illustrative images of the primary and external packaging of a Transfer, in the form in which it will be commercialized.

TRANSFERS' PRIMARY PACKAGING

TRANSFERS' EXTERNAL PACKAGING

4. DEVICE RECOMMENDATIONS, PURPOSE, OR INTENDED USE

It is recommended that Open Tray Transfers be attached to the implant or prosthetic component that is to be molded using direct molding techniques, thereby transferring the exact position of the implant and/or prosthetic component to the laboratory model.

It is recommended that Closed Tray Transfers be attached to the implant or prosthetic component that is to be molded using indirect molding techniques, thereby transferring the exact position of the implant and/or prosthetic component to the laboratory model.

5. PRINCIPLES OF INTENDED DEVICE FUNCTION AND PERFORMANCE

Transfers are devices used to transfer the position of the implant and/or prosthetic component to the laboratory model.

Transfers are fitted and fixed onto the Plenum Implant using a Morse Taper connection, a double-inlet trapezoidal thread profile, and a self-threading tip. The transfer is fitted and fastened onto the component using a conical screw thread and connection.

6. INSTRUCTIONS FOR USE

- Carefully read the instructions for use, recommendations, and guidelines prior to using the Transfers;
- Device supplied in sterile conditions and intended for use by trained, qualified dental surgeons with experience and knowledge of implant dentistry techniques;
- Before using the transfer, the dental surgeon must strictly verify that the device has the same prosthetic transfer/Implant and/or transfer/component interface that has been installed.

HANDLING:

- When carrying out the implant or component molding procedure, use the corresponding transfer according to appropriate protocols described in the literature;

ATTENTION: Figures are merely illustrative. They do not represent the actual dimensions and characteristics of the device.

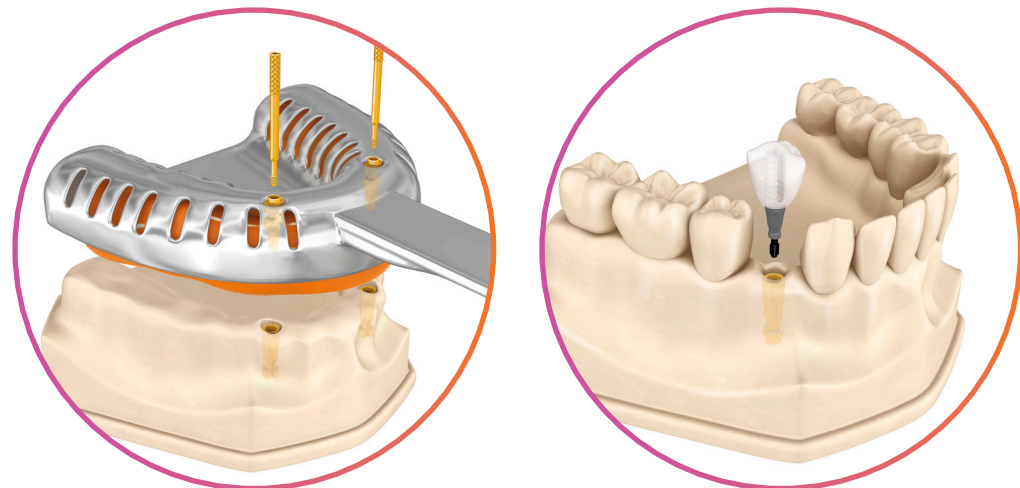
OPEN TRAY:

- Gently fit the transfer over the implant or component with the flaps positioned in the vestibular and lingual regions;
- With the transfer positioned stably, fasten it onto the component or implant. Do not exceed a force of 10 Ncm;
- In case of multiple rehabilitation, transfers must be fastened together using acrylic resin;
- Once transfers are fixed in the ideal position, select the most suitable molding material and make an impression of the mold;
- After the molding material has been attached, completely loosen the screw and remove the tray;
- Verify whether the transfer has shaped the mold. If the transfer has not been captured by the molding material, repeat the process;
- Once the transfer has attached to the mold, carefully fit and fasten the implant's analog or component without moving the part;

- Empty the molding material to simulate the patient's soft tissue, then subsequently pour the plaster over the mold, until it covers the entire analog;
- After the plaster curing period, remove the mold from the plaster. Carefully unscrew the transfer from the analog;
- After preparation of the prosthesis, assess its adaptation onto the respective analog;
- Refer to the torque indicated for the prosthetic component to be used for correct prosthesis preparation. Excessive torque can produce unsatisfactory results and poor adaptation of the system.

CLOSED TRAY:

- Gently fit the transfer over the implant or component with the flaps positioned in the vestibular and lingual regions;
- With the transfer positioned stably, fasten it onto the component or implant. Do not exceed a force of 10 Ncm;
- In case of multiple rehabilitation, transfers must be fastened together using acrylic resin;
- Once transfers are fixed in the ideal position, select the most suitable molding material and make an impression of the mold;
- Loosen the Transfer screw and remove it from the component or implant and fasten it onto the analog;
- Fit the transfers fastened to the mold's analog, paying particular attention to the initial position of the transfers during the printing of the mold;
- Empty the molding material to simulate the patient's soft tissue, then subsequently pour the plaster over the mold, until it covers the entire analog;
- After the plaster curing period, remove the mold from the plaster. Carefully unscrew the transfer from the analog;
- After preparation of the prosthesis, assess its adaptation onto the respective analog;
- Refer to the torque indicated for the prosthetic component to be used for correct prosthesis preparation. Excessive torque can produce unsatisfactory results and poor adaptation of the system.

ATTENTION: Figures are merely illustrative. They do not represent the actual dimensions and characteristics of the device.

7. PRECAUTIONS

- Transfers are dental devices and must only be used by health care and dental professionals once they have carefully read the instructions for use and the information provided on the packaging. All information on the use of instruments and the warnings, precautions, and recommendations included in these instructions for use must be taken into consideration. A failure to observe these points may result in complications during procedures;
- Use of these devices is restricted to dental applications. Transfers must only be used by properly trained and qualified professionals with proven experience and knowledge of implant dentistry techniques and installation of dental prostheses onto implants;
- The transfers' device labels include the date of manufacture, device code, batch code, basic dimensions, ANVISA (Brazilian National Health Regulatory Agency) device registration number, technical product manager, company information, and the respective symbology that allows the instrument to be correctly identified and traced;
- Correspondence between the diameter of the implant installed in the patient and the diameter of the transfers that will be used must be carefully verified;
- The use of inappropriate techniques, procedures, and inadequate surgical conditions or planning may cause harm to the patient and compromise the corresponding implant dentistry procedure;
- For safety and effectiveness and not to compromise the performance of the intended use, only prosthetic components and implants manufactured by Plenum should be used. Components and/or implants manufactured by other companies may result in dimensional incompatibility, improper fitting, micro-movements and increase the risk of corrosion (electrolytic or galvanic) and fracturing of these devices, resulting in dimensional incompatibility between parts and early failure of the implants and components, compromising the result of the surgical procedure due to loss or fracture of the implants or failure of the implant/prosthetic component system;
- Patients must be informed of all limitations, risks, complications, warnings, and potential adverse effects arising from the use of prosthetic components, as well as from the oral implantology procedure;
- Before each procedure, ensure that there is correct fitting between the parts and that the respective components are compatible with the selected transfers;
- Improper selection of components, as well as errors in the indicated use, handling, and installation of the device, may result in damage to the device's structure, which may contribute to failure due to fatigue, fracturing, or loosening of the component;
- The reprocessing or reuse of these devices causes changes in physical/chemical and mechanical characteristics with macro and microstructural damage to the device. These alterations may impair the device's functionality and not guarantee its safety and effectiveness. Such alterations also void any warranty for related devices;
- The Plenum device sequence must always be used during procedures. Instrumentals and/or prosthetic components from other manufacturers do not guarantee optimal performance during the intended use of the Plenum Implant System.

8. WARNINGS

- Single-use only, do not resterilize or reuse.
- Do not use if the package is opened or damaged or if the expiration date has been exceeded;

9. CONTRAINDICATIONS

Since they are intended for transitory use, transfers do not present contraindications and side effects in patients. However, the Transfers must be used by trained professionals that are specialized in the area and are knowledgeable in the techniques of handling this type of component, exclusively for the specific purpose for which it was designed.

Dental surgeons must prudently assess the patient's condition before surgery and the use of the Transfer. In this case, contraindications in Transfers involve:

- Inflammation and/or periodontal infection;
- Patients presenting signs of allergy or hypersensitivity to the composition of the raw materials used in manufacturing Plenum Dental Implants (see "Device Composition").

10. POSSIBLE ADVERSE EFFECTS

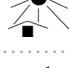
Every surgical procedure presents risks and possibilities of complications, such as infections, hemorrhages, allergic drug reactions and anesthetic risks, among others. Possible adverse effects associated with the use of transfers may include, but are not limited to:

- Improper choice of model or device size, or the use of inappropriate techniques may make the transfer procedure unfeasible;
- Allergic reaction to the transfer's raw material (see "Device Composition");
- The patient must be informed and understand that in all surgical procedures, there is the possibility of complications. The patient must be informed by the dental surgeon of all these complications, and of the risks resulting from implant surgery.

11. SYMBOLS DESCRIPTION

The Table 2 describes the symbols that may be on the device labeling. Please check the physical labeling for the symbols applied to the device. The symbols express information related to the identification and use of the device.

TABLE 2 – Symbols used for labeling to express information related to the identification and use of the device.

	Manufacturer
	Country and date of manufacture
	Use-by date
	Batch code
	Catalog number
	Sterilized using ethylene oxide
	Sterilized using Irradiation
	Sterilized using vaporized hydrogen peroxide
	Non-sterile
	Do not re-sterilize
	Do not use if package is damaged
	Single sterile barrier system
	Double sterile barrier system
	Single sterile barrier system with protective packaging inside
	Single sterile barrier system with protective packaging outside
	Keep away from sunlight
	Keep dry
	Lower limit of temperature
	Upper limit of temperature
	Temperature limit
	Do not re-use
	Consult electronic instructions for use (plenum.bio/ifu)
	Unique device identifier
	Device manufacturing material
	Caution: Federal law restricts this device to sale by or on the order of a physician or dentist. (Notification required by FDA for United States market)
	Reprocessing prohibited (Brazilian Requirement)

NOTE: The symbols related to the labeling meet the requirements established in the ISO 15223-1:2021 standard.

12. STERILIZATION

- Transfers are supplied sterile and sterilized using gamma irradiation;
- Single-use only, do not reprocess, re-sterilize or reuse;
- Before use, check the integrity of the packaging and the expiration date. Do not use the device if the package is violated or with the sterilization expiration date expired;
- Date of manufacture, expiration date, device code, batch code and other information regarding identification and traceability are on the device labeling;
- To maintain sterility, the device must be opened only at the time of use and handled in appropriate and aseptic environments;
- The opening of the packaging for the use of the device must only be done by qualified and trained personnel to perform this procedure.

13. EXPIRATION DATE

- Indicated on device labeling.
- Before use, check the expiration date, do not use the device if the expiration date has been exceeded.

14. IDENTIFICATION AND TRACEABILITY

To ensure the traceability of transfers and comply with health surveillance requirements, the dental surgeon or other staff must keep the information regarding the device used in the patient's medical record. Additionally, this information must also be passed on to the device distributor and patient, to complete the traceability cycle. The information necessary in traceability is related to the dental implant used and surgery, as follows:

- Patient's name;
- Surgeon's name;
- Hospital/clinic/office identification;
- Manufacturer and distributor identification;
- Date of surgery;
- Device code;
- Batch code;
- Quantity;
- ANVISA device registration number.

The responsible dental surgeon and their team must make use of the information contained in the three (03) traceability labels supplied with transfers, fixing this information in the patient's clinical record to maintain traceability. In addition, this information must be provided to the patient so that they have information about the component used in their procedure. Labels contain the following information necessary for the traceability of the transfer used.

- Manufacturer identification;
- Device name or commercial model;

- Device code;
- Batch code;
- ANVISA device registration number.

Traceability information is necessary for notification by the health service and/or by the patient to the National Health Surveillance Agency – ANVISA and to the manufacturer when complaints, technical complaints and/or adverse events occur for the conduct of the appropriate investments.

15. STORAGE, TRANSPORT, CONSERVATION AND HANDLING CONDITIONS

- Do not use if package is opened or damaged or if expiration date has been exceeded. If the packaging is damaged, the device must be considered non-sterile and discarded;
- The device must be stored and transported in a clean environment, protected from heat or humidity sources, away from the incidence of sunlight, and free from the action of weathering, in order to prevent any damage or change in the packaging or its physical form that makes it impossible to use;
- The storage of the device on the shelves or drawers/ cabinet must be in such a way that facilitates the visualization and manipulation of the operator. Do not store the device near lamps, so that the information contained in the packaging is not affected;
- The device should only be opened at the time of use in a sterile surgical field;
- The device handling should only be done by qualified and trained personnel to perform this procedure.

16. DISPOSAL OF DEVICE

- Discard the device if any of the successive packages that ensure sterility is damaged or opened;
- After the device is used or removed from the patient, transfers that are not subject to analysis or subsequent studies must be disposed of by the health service establishment (hospitals, offices, and clinics);
- It is recommended that the device be completely disassembled to ensure that it cannot be reused. The health service institution/establishment is solely responsible for disassembling the device;
- After disassembly, the instrument must be identified as being unsuitable for use and disposed of in accordance with health service waste management standards (hospitals, offices, and clinics) and applicable environmental legislation;
- Transfers that are not used during the procedure must also be discarded following the guidelines above.

17. CUSTOMER SERVICES

- If the device is subject to any adverse event that presents a risk of harm to the patient, generates or has the potential to cause injury or a threat to public health, is out of specifications, or is producing unsatisfactory results,

the respective health professional must generate a technical report detailing the specific instance of non-compliance, including claimant information, device code and batch code, and patient data; directly notifying Plenum's Customer Service;

- If necessary, send the clean, decontaminated, sterilized, packaged, and correctly identified device, with the description of non-conformities, to the following address: M3 Health Indústria e Comércio de Produtos Médicos, Odontológicos e Correlatos S.A. Avenida Ain Ata, N° 640 – Lote 15 – Quadra B – Jardim Ermida I, Multivias II – Polo Industrial e Logístico – CEP 13212-213 – Jundiaí – SP – Brazil. Telephone: +55 11 3109-9000; Email: customerservice@plenum.bio
- ANVISA also provides a channel to receive notifications of incidents, adverse events and technical complaints related to the use of devices and services under health surveillance, which can be found on the website of the National Health Surveillance Agency – ANVISA, at the electronic address www.anvisa.gov.br, using the NOTIVISA link.
- FDA also provides a channel available to report incidents, adverse events, and technical complaints related to the use of devices and services subject to health regulations, which can be found in the FDA Manufacturer and User Facility Device Experience (MAUDE) database.

18. LIMITED WARRANTY

- The devices manufactured by Plenum have a factory warranty against manufacturing defects. The occurrence of defects in the device must be reported immediately to the manufacturer, observing the legal deadline, and attaching a technical report with the details of the non-compliance with the device. This report must be generated and signed by a licensed and qualified health professional, also observing the standards belonging to ANVISA (see Customer Service) in case of risk to the patient's health.
- The Warranty of devices manufactured by Plenum is conditioned to the strict observation and follow-up of the indications for use, contraindications, precautions, warnings and other information contained in the instructions for use. The lack of such compliance and follow-up exempts the manufacturer and distributor of the devices from any liability, in particular in the following cases:
 - Reprocessing, resterilization, or reuse of the device by the user;
 - Improper use or use other than in a surgical setting;
 - Device not used according to instructions for use.

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